



# NASAL HIGH FLOW OXYGEN THERAPY ON THE DRAEGER INFINITY V500

## Consumables for Patients requiring oxygen 2- 25L/min

- RT 330 F & P Optiflow Tubing kit.



Note: Remove Pressure relief valve from pack as its an open circuit hence not required and Draeger ventilator have inbuilt pressure relieve valve.

- Use 'single' limb circuit.
- Do not use safety valve. (not required).
- OPT316 F & P Infant optiflow junior cannula ( butterfly) 
- OPT318 F & P Paediatric opti flow junior cannula ( bird) 

## Consumables for patients requiring oxygen 20-50L/min

- RT202 F & P Adult Optiflow tubing kit.



- Nasal cannula Small OPT942
- Nasal cannula Medium OPT944
- Nasal cannula Large OPT946

